

**DENTAL COTTON®**

Dental cotton rolls are used to reduce the saliva flow inside the mouth, keep it open and easily accessible for dental operations.

Instructions for Use:

- * For the instance of dental cotton, dispense the cotton from its original packing using a cotton tweezer, and with the same implement hold the cotton and place it into the patient's mouth in the desired location.
- * For the instance of the medical cotton, dispense the cotton from its original packing and use it according to the specific necessity: Bandage, injectology or orthopedic. Do not use for open wounds.
- * Once the dental or medical procedure is finished, throw it away immediately.

Hazard Statements/ Precautionary Statements**Warnings and Precautions**

For single use only.

Storage Conditions

Keep tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight, humidity, sources of ignition. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children.

Packaging

·PRD.01.20020 1x60 pcs

**DENTAL COTTON®**

Dental Cotton ruloları, ağız içindeki salıva akışını azaltmak ve operasyonlar sırasında çalışılan bölgeyi açık ve ulaşılabilir tutmak için kullanılır.

Kullanım Endikasyonları:

- * Pamuk cımbızı yardımıyla pamuğu orijinal kabından ayırın, aynı cımbız ile hasta ağızında istenilen bölgeye pamuğu yerleştirin.
- * Medikal pamuğu ihtiyaca göre bandaj, enjeksiyon veya ortopedik amaçlara yönelik kullanabilirsiniz. Açık yaralarda kullanmayınız.
- * Dental prosedür bitiminde mevzuata göre bertaraf ediniz.

Tehlike İfadeleri/ İhtiyati İfadeler**Uyarılar**

Sadece tek kullanımlıktır.

Saklama koşulları

Kuru ve serin yerde 4-28°C ağız kapalı olarak saklayınız. Güne ışığı, nem ve tutuşturucu materyallerden uzak tutunuz. Son kullanma tarihinden sonra kullanmayınız. Ulusal mevzuatlar gereğince bertarafını sağlayınız. Çocukların erişemeyeceği yerde muhafaza ediniz.

Paketleme

* PRD.01.20020 1x60 adet

**PRESIDENT DENTAL GmbH**

Kesselbodenstrasse 5, 85391 Allershausen, Germany
Tel: +49 8166 389 9 820 • Fax: +49 8166 389 9 821
www.presidentdental.com • info@presidentdental.com