DE

Prebond SE®

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Prebond SE®

Instruction for use

Prebond SE Self Etch is a self-etching, light curing one component adhesive for the adhesive restorative technique.

Composition

Prebond SE contains functional MDP monomer, methacrylate, photoinitiators, ethanol, water.

Other areas of use

Can be used together in the following areas:

- With hypersensitivity treatments
- With Root Surface sensitivity treatments
- With Cavity and Dentin Varnishes
- With Fit and Fissure Sealants

Indications for use

Adhesive for the adhesive restoration technique used on direct and indirect restorations.

- Fixing of composite-material and compomer at enamel and dentin

- adhesive fixing of restorations of ceramic or composite at enamel and dentin

- Fixing and repairing of light cure composites at hybrid ceramic, composit, metal or amalgam

- also usable as dentin-sealer

Side effects

Side effects with this medical product are extremely rare if it is properly prepared and applied. Should you encounter undesirable side-effects – even in case of doubt – then please contact us and give a precise description of the circumstances and symptoms. Constituent parts of Prebond SE can lead to sensitivity or an allergic reaction with a predisposed person.

Working instruction

Ensure that the processing instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions have been observed, then please inform us and give a precise description of the circumstances.

Interactions

Phenolics and other substances (e.g., zinc-oxide-eugenol), which may affect the polymerization, must not be contacted with Prebond SE Self Etch.

Use the restoration-material according to the instructions of the manufacturer.

4. Sealing of the dentin

Please take note of the working instruction at point 1.1 and 1.2.

Emergency-measures

Acute danger at the patient throughout accidentally swallowing is in consideration of the small quantities (application with disposable brush) not to be expected. If there is a direct contact with the oral mucosa rinse carefully with tap water. In case of eye-contact rinse carefully with tap water. For evaluation and after treatment contact an ophthalmologist.

Toxicological effect

The toxicity of the direct the vital dentin contacting and accordingly used Prebond SE which penetrates the dentin canals is comparable with glassionomer cement or polycarboxylat.

Restoration

1. Conventional filling-technique

a) Preparation of the cavity

According to the principles of the restorative-technique. If necessary a punctional capping of the pulp with a strong-setting calciumhydroxide-cement is to be set.

Conditioning of the enamel with a finishing brush. Unpolished enamel is to be etched with etching-gel or cleaned with pumice. Afterwards rinse enamel well and dry it. The use of dam is recommended for hygienic reasons.

b) Application

Shake bottle well before use and give one drop into a dispensing well.

Apply Prebond SE for 20 s to dentin and enamel. Dry gently with oil-free air the Prebond SE Self Etch for 5 s.

Afterwards light-cure for 10 s (halogene- or LED-polymerisation-device with a light-intensity of >800 mW/cm²). If the light-intensity is low the light-curing prolongs correspondingly.

c) Fillings of composite and compomer

Use restoration material according to the instruction of the manufacturer.

2. Fixing of inlays/onlays, veneers and crowns

While using Prebond SE it is to observe that enough light (according to the instructions of the manufacturer) is getting into the complete restoration.

a) Preparing of the restoration

Prepare the restoration concerning to the working instructions of the corresponding substance.

b) Preparation of the cavity

As described in point 1.1.

c) Application

As described in point 1.2.

d) Cementation

According to the instructions of the manufacturer.

3. Adhesion of other surfaces

a) Preparation

b) Ceramics

Intraoral application:

Ceramic surfaces should be roughened with a diamond bur, sandblasted or etched with 37% phosphoric-acid rinsed and dried with oil-free air.

Extra oral application:

Ceramic surfaces can be etched with 5% HF acid afterwards rinsed and dried with oil-free air.

Optional:

Silnization of ceramic-surfaces (according to instructions of manufacturer)

c) Metal and Amalgam

Roughen surfaces with a diamond bur or sandblast them. Afterwards clean with stream jet (extra oral) or acetone and dry with oil free air.

d) Composite

Surfaces of composites are to be roughened with diamond bur or to be sandblasted, cleaned with phosphoric acid, rinsed and dried with oil free air.

e) Application

Shake bottle well before use and give one drop into the dispensing well. Close the bottle immediately after use. Apply Prebond SE Self Etch with a disposable brush for 20 s onto the surface. Dry the bonding gently for 5s with oil free air. Afterwards light-cure for 10 s (halogene- or LED Polymerisation-device with a light-intensity of >800mW/cm²).

If the light-intensity is lower the light-curing prolongs correspondingly.

f) Restoration

Use the restoration-material according to the instructions of the manufacturer.

4. Sealing of the dentin

Please take note of the working instruction at point 1.1 and 1.2.

Emergency-measures

Acute danger at the patient throughout accidentally swallowing is in consideration of the small quantities (application with disposable brush) not to be expected. If there is a direct contact with the oral mucosa rinse carefully with tap water. In case of eye-contact rinse carefully with tap water. For evaluation and after treatment contact an ophthalmologist.

Toxicological effect

The toxicity of the direct the vital dentin contacting and accordingly used Prebond SE which penetrates the dentin canals is comparable with glassionomer cement or polycarboxylat.

Restoration

1. Conventional filling-technique

a) Preparation of the cavity

According to the principles of the restorative-technique. If necessary a punctional capping of the pulp with a strong-setting calciumhydroxide-cement is to be set.

Conditioning of the enamel with a finishing brush. Unpolished enamel is to be etched with etching-gel or cleaned with pumice. Afterwards rinse enamel well and dry it. The use of dam is recommended for hygienic reasons.

b) Application

Shake bottle well before use and give one drop into a dispensing well.

Apply Prebond SE for 20 s to dentin and enamel. Dry gently with oil-free air the Prebond SE Self Etch for 5 s.

Afterwards light-cure for 10 s (halogene- or LED-polymerisation-device with a light-intensity of >800 mW/cm²). If the light-intensity is low the light-curing prolongs correspondingly.

c) Fillings of composite and compomer

Use restoration material according to the instruction of the manufacturer.

2. Fixing of inlays/onlays, veneers and crowns

While using Prebond SE it is to observe that enough light (according to the instructions of the manufacturer) is getting into the complete restoration.

a) Preparing of the restoration

Prepare the restoration concerning to the working instructions of the corresponding substance.

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