Kullanma Talimatı

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Instructions for use

Procem is an eugenol-free easily to process cement. It does not influencethe setting of resin-based filling and luting materials. It is recommended to use **Procem** in case of allergies to eugenol. To support the preventies of tooth vitality **Procem** contains calcium hydroxide and hydroxylapatit.

Field of Application

Procem is used as a temporary cement for attachment and sealing.

Method of use

Procem should be used at room temperature. In case of lower temperature (use of a cooled glass-plate) the working time will be extended correpondingly. Base and catalyst paste should be mixed in proportion: 2 : 1 v/v. For that purpose, put one length of the catalyst paste and accordingly double the length of base on a mixing pad of strong, non-absorbing material or glass-plate respectively (1 : 1 w/w). Then mix the two pastes for approximately 30 seconds until the material takes on a homogeneous consistency. Use a plastic spatula. Apply a thin layer inside the temporary crown or bridge and affix to the prepared tooth. Do not use dry cores, moisten them with some water beforehand. **Procem** remains workable for about 1-2 minutes and sets in the mouth within 3-6 minutes. 3-5 minutes after **Procem** has set, remove any excess.

Supplementary indications

To obtain a softer material and to keep **Procem** workable for a longer time, add a little bit more of **Procem**-base or a small trace of Vaseline. A humid milieu (humid cores) additionally facilitates to remove work afterwards. A harder and faster setting material is obtained by adding more catalyst paste (temporary sealing), or zinc oxide. Extensive superficial and multisurfaced cavities as well as temporary longterm sealing work (max. 1 month) should be treated with a harder cement. Residues of **Procem** can be removed from crowns easily after they have been shortly sprayed with a cooling spray.

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