ZENIT color

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■ Made
■ in
■ Germany
-best quality
-high standards

EN

Range of Application

Characterization of composite facings on crowns and bridges, individualization of tailor-made resin teeth as well as characterization of denture base resins in partial and full dentures. Available in ten different shades.

Instructions

The stains can be applied directly from the syringe. Alternatively, the stains can be put on a palette for mixing and applied with a thin brush. All shades are intermixable. Dialog Chroma-Flow can also be mixed with composite pastes. Please avoid introducing bubbles into the mixture and consider the instructions for the composite. Apply dialog Chroma-Flow in layers no thicker than 0.3mm.

Polymerization

Dialog Chroma-Flow can be polymerized in all conventional light-curing systems. Before the final polymerization, the complete polymerization site may be covered with expers-Gel. This avoids the formation of a new dispersion layer and facilitates the finalization of the object. Use a light-curing system with an emission spectrum of 310-500 nm. Perfect physical properties can only be obtained with immaculate lamps. It is important to test the light intensity regularly, in line with the manufacturer's instructions.

Finishing

All silicone polishers (square-edge or knife-edge wheels, cylinder) as well as hard metal and diamond mills are suitable for finishing the material. The facing is polished with handpiece-mounted goat-hair brushes, woolen wheels and a polishing paste. Careful conditioning of the surface is essential if optimal results are to be achieved, thus virtually precluding deposits (nicotine, caffeine etc.) which could cause discoloration.

Working Range Of The Material

1-3 minutes, depending on the light conditions.

Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been.

Composition Filler, Anorganic

(50 weight % respectively 36% by volume)
Glass filler (medium particle size: 0.7 µm)
Pyrogenic silica (medium particle size: 0.01 µm)

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"Follow President Dental Product Clinical Cases On [f](#) President Dental Study Club"